
SPIRO®

REF 702 CE 0402

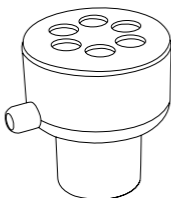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en Instructions for Use

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


no Bruksanvisning

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sv Bruksanvisning

FOGLESS
International

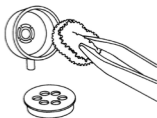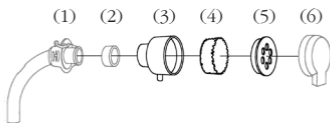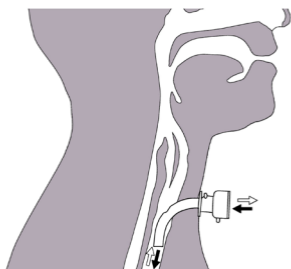CE LOT REF **Rx Only**

| | | |
|----|--------------------------|----|
| da | Brugervejledning | 2 |
| de | Gebrauchsanweisung | 4 |
| en | Instructions for Use | 6 |
| es | Instrucciones de uso | 8 |
| fi | Käyttöohjeet | 10 |
| fr | Mode d'emploi | 12 |
| it | Istruzioni per l'uso | 14 |
| nl | Gebruiksaanwijzing | 16 |
| no | Bruksanvisning | 18 |
| pl | Instrukcja obsługi | 20 |
| pt | Instruções de utilização | 22 |
| sv | Bruksanvisning | 24 |

SPIRO FUGTER **REF** 702

Kun en-patients brug

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SPIRO HUMIDIFIER **REF** 702

Single patient use only

The HUMIDIFIER has an exchangeable filter and is used on a tracheostomy cannula to take advantage of the heat and moisture that is exhaled. It should be replaced with a new unit after 29 days of use, or more often if necessary.

N.B. THE HUMIDIFIER MUST NEVER BE USED WITHOUT THE FILTER

DAILY CHECKS: Check humidifier operation daily and inspect for damage. Replace humidifier with new unit in case of damage or defective operation.

CLEAN the valve HOUSING (3) daily. Do not use sharp tools. Use the fingers or PLASTIC TWEEZERS (P/N 705, sold separately) to remove/re-insert the filter. The valve housing and COVER (5) shall be air dried after washing with luke-warm water and mild fragrance-free detergent. Other detergents or solvents must not be used. Always check the operation again after cleaning and check that the air passages are free. Do not wash and re-use the filter!

REPLACE THE FILTER (4) (P/N 740, sold separately) as required, at least twice every 24 hours. Replacement may be required more often, perhaps several times per hour, if a build-up of mucus or phlegm is present. Ensure that the filter is seated evenly against the bottom of the filter housing, and that the air flow through the filter is not obstructed by phlegm or other substances.

A HEAT SHIELD/SHOWER PROTECTOR (6) (P/N 703, sold separately) should be used as required with the inhalation opening facing downward.

An ADAPTER (2) (P/N 707, 708, sold separately) is only required for connection to a cannula (1) without an ISO 15 connector (taper).

SUPPLEMENTAL OXYGEN can connect via the Ø5.5mm nipple on the valve housing.

RESISTANCE TO FLOW for SPIRO Humidifier is < 0.8 cm H₂O* and with heat shield/shower protection on it has a resistance to flow < 1.2 cm H₂O*.

The humidifier must be removed before inhalation of medicine.

All products should be stored in a dark and dry environment and at normal indoor temperature.

For the STORAGE PERIOD of SPIRO PRODUCTS: See year and month next to the 'HOURLASS' symbol on the product label. The products shall be disposed of in accordance with the applicable national regulations.

ALWAYS CONSULT YOUR DOCTOR, SUPPLIER OR MANUFACTURER IF YOU HAVE ANY QUESTIONS.

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Fogless International AB