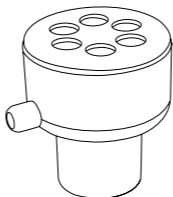
SPIRO®

HUMIDIFIER

REF 702 CE 0402

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en Instructions for Use

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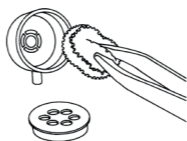
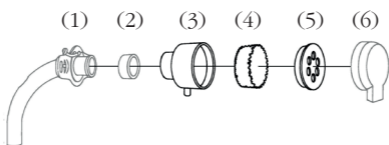
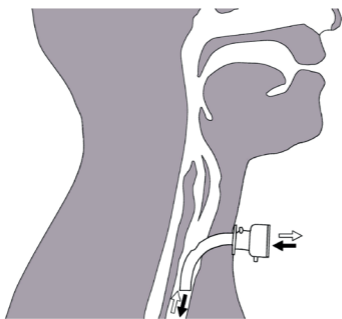
no Bruksanvisning

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sv Bruksanvisning

Symbol Glossary

Symbol	Title	Description
	Caution!	Indicates the need for the user to consult the Instructions for Use for important cautionary information, such as warnings and precautions.
	Consult instructions for use	Indicates the need for the user to consult the Instructions for Use.
	Manufacturer	Indicates the medical device manufacturer.
	Catalogue number	Indicates the manufacturer's catalogue number so that the medical device can be identified.
	Use-by date	Indicates the date after which the medical device is not to be used.
	Batch code	Indicates the manufacturer's batch code so the batch or lot can be identified
	Non-sterile	Indicates a medical device that has not been subjected to a sterilization process.
	Keep away from sunlight	Indicates a medical device that needs protection from light sources.
	Keep dry	Indicates a medical device that needs to be protected from moisture.

Symbol	Title	Description
	Do not use if package is damaged	Indicates a medical device that should not be used if the package has been damaged or opened.
	Do not re-use	Indicates a medical device that is intended for one use. (Article 740 only)
	Quantity	Indicates the total number of devices in the package.
QTY:	Quantity	Indicates the total number of devices in the package.
Rx Only	Prescription only (USA)	Indicates that the product is a medical device as defined in 21 CFR 820.3(l) and Federal Law (USA) restricts this device to sale by or on the order of a physician (21 CFR 801.109)
	CE mark	Indicates the manufacturer declaration that the product complies with the essential requirements of the relevant European health, safety and environmental protection legislation.

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SPIRO HUMIDIFIER **REF** 702

Single patient use only

The HUMIDIFIER has an exchangeable filter and is used on a tracheostomy cannula to take advantage of the heat and moisture that is exhaled. It should be replaced with a new unit after 29 days of use, or more often if necessary.

N.B. THE HUMIDIFIER MUST NEVER BE USED WITHOUT THE FILTER

DAILY CHECKS: Check humidifier operation daily and inspect for damage. Replace humidifier with new unit in case of damage or defective operation.

CLEAN the valve HOUSING (3) daily. Do not use sharp tools. Use the fingers or PLASTIC TWEEZERS (P/N 705, sold separately) to remove/re-insert the filter. The valve housing and COVER (5) shall be air dried after washing with luke-warm water and mild fragrance-free detergent. Other detergents or solvents must not be used. Always check the operation again after cleaning and check that the air passages are free. Do not wash and re-use the filter!

REPLACE THE FILTER (4) (P/N 740, sold separately) as required, at least twice every 24 hours. Replacement may be required more often, perhaps several times per hour, if a build-up of mucus or phlegm is present. Ensure that the filter is seated evenly against the bottom of the filter housing, and that the air flow through the filter is not obstructed by phlegm or other substances.

A **HEAT SHIELD/SHOWER PROTECTOR** (6) (P/N 703, sold separately) should be used as required with the inhalation opening facing downward.

An **ADAPTER** (2) (P/N 707, 708, sold separately) is only required for connection to a cannula (1) without an ISO 15 connector (taper).

SUPPLEMENTAL OXYGEN can connect via the Ø5.5mm nipple on the valve housing.

RESISTANCE TO FLOW for SPIRO Humidifier is $< 0.8 \text{ cm H}_2\text{O}^*$ and with heat shield/shower protection on it has a resistance to flow $< 1.2 \text{ cm H}_2\text{O}^*$.

The humidifier must be removed before inhalation of medicine.

All products should be stored in a dark and dry environment and at normal indoor temperature.

For the STORAGE PERIOD of SPIRO PRODUCTS: See year and month next to the 'HOURLASS' symbol on the product label. The products shall be disposed of in accordance with the applicable national regulations.

ALWAYS CONSULT YOUR DOCTOR, SUPPLIER OR MANUFACTURER IF YOU HAVE ANY QUESTIONS.

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Fogless International AB